KENDALL

SHARPS-A-GATOR*

SHARPSAFETY* Sharps Collection & Disposal System

SHARPSAFETY* Sharps Collection & Disposal System — DIRECTIONS FOR USE

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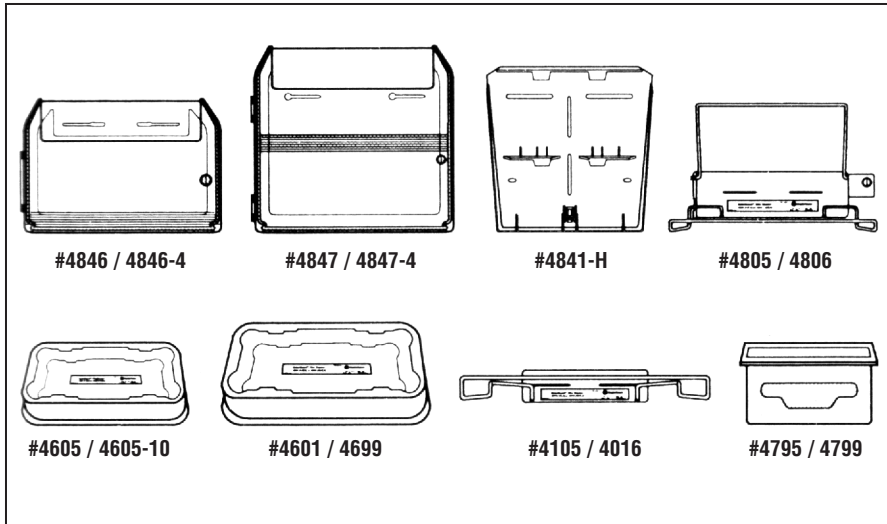
4815 Series GATORGUARD* Jr. and 4835 & 4836 Series GATORGUARD* Sharps Container Holders

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Read all precautions and instructions before installing and using the GATORGUARD Jr. and GATORGUARD Sharps Container Cabinets, Hidden Bracket, Wire Brackets, GATORGUARD Jr. and GATORGUARD Table Top Holders. Improper installation or use may result in exposure to biohazardous materials and percutaneous or other injuries, or damage to cabinet.

PRECAUTIONS

- Never drop sharps into empty cabinet.
- Keep cabinet (4846 & 4847) or wire bracket with lock (4805/4806) locked at all times except when installing or removing sharps container.
- Use only with 4815 Series GATORGUARD Jr. or 4835 & 4836 Series GATORGUARD Sharps Containers. (The 4841-H can only be used with the 4835 Series.)
- Never hold used sharps container below recommended fill line.

INSTALLATION INSTRUCTIONS

Tools Needed: screwdriver, hammer, level, drill bit and drill, wall anchors, tape measure, pencil, screws and washers.

The GATORGUARD Jr. and GATORGUARD cabinets, wire brackets and hidden bracket are designed to be installed on any wall surface using your existing unit wall holes (attachment points) or any new wall surface. After determining wall surface, make sure area behind wall is clear of all electrical wires and plumbing connections before drilling into wall.

To install cabinets (4846 or 4847) by replacing existing unit:

Loosen top screws or bolts and remove bottom screws or bolts, then slide cabinet into position on top screws or bolts using keyhole slots. If necessary, mark new bottom mounting points, remove cabinet and follow (d & e) in "To Install Cabinets (4846 or 4847), hidden bracket (4841-H) or wire brackets (4015/4016 or 4805/4806) for the first time." If possible drill new bottom holes with more vertical distance between them to ensure greater stability for the cabinet.

To install cabinets (4846 or 4847), hidden bracket (4841-H) or wire brackets (4015/4016 or 4805/4806) for the first time:

- a. Select a mounting position 50" (± 4") from top slots of cabinet, hidden bracket or wall brackets to floor that maintains adequate clearances for:
 - Height of cabinet, hidden bracket or wire brackets with inserted container (clearance of at least 8").
 - Swing of cabinet door (clearance of at least 4") for cabinet only.
 - Optional bottom mounted glove box for cabinet only (clearance of at least 6").
- b. Using a level, mark a horizontal line on the wall at desired mounting position (at the top slots of cabinet, hidden bracket or wire brackets).
- c. For cabinet only, open door and use cabinet back as a template to mark points for wall anchors. Space mounting points as far apart as possible vertically and horizontally.
- d. Drill hole in wall to accommodate wall anchor. Use hammer to tap in wall anchor flush with wall surface.
- e. Holding cabinet, hidden bracket or wire brackets in position on wall, affix with screws and washers. Do not overtighten or damage to cabinet, hidden bracket or wire brackets may occur.

To install table top holder:

Never place sharps container on floor or any other surface without table top holder.

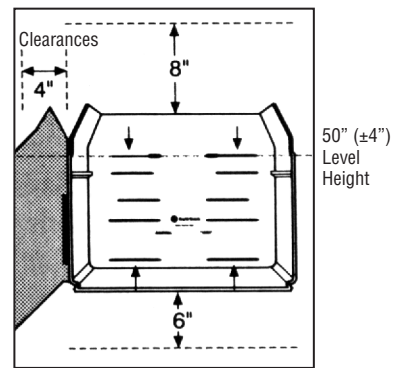
- a. Remove adhesive strip from bottom of holder and secure to appropriate surface.

To install optional glove box to cabinet:


- a. Slide left side of glove box into left slot on bottom of cabinet.
- b. Lift right side of glove box up until it snaps into position on bottom of cabinet.

To remove glove box from cabinet:


- c. Locate "Release Tab" on right side of glove box. Push "Release Tab" in while pulling down until glove box releases from bottom of cabinet. As you push the "Release Tab" in, place your opposite hand under the glove box for stability.


4846 GATORGUARD Jr. Cabinet

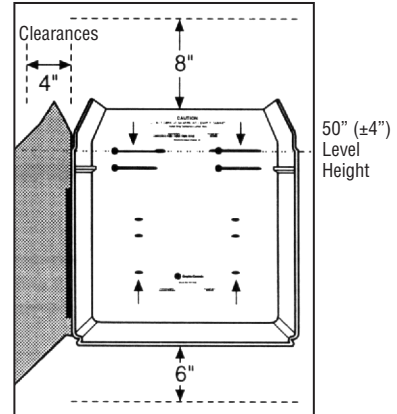
4847 GATORGUARD Cabinet

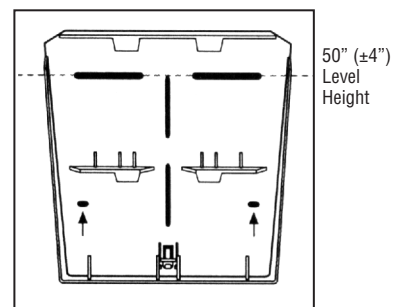
4841-H Hidden Bracket

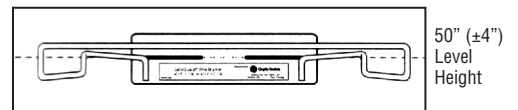
4015 / 4016 GATORGUARD Wire Bracket

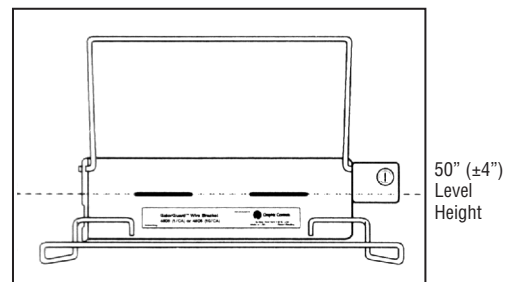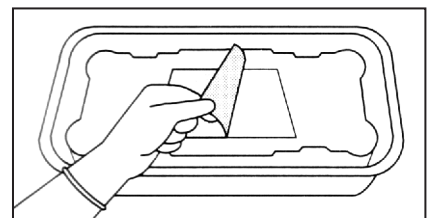4805 / 4806 GATORGUARD Wire Bracket with Lock

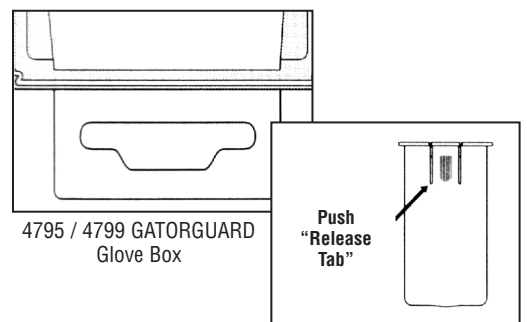4605 / 4605-10 GATORGUARD Jr. Table Top Holder or 4601 / 4699 GATORGUARD Table Top Holder

4795 / 4799 GATORGUARD Glove Box

Side View

SHARPS CONTAINER INSERTION AND REMOVAL INSTRUCTIONS

For safe operation, the GATORGUARD Jr. or GATORGUARD Sharps Containers must be properly installed in the GATORGUARD Jr. or GATORGUARD Sharps Container Cabinets, Hidden Bracket, Wire Brackets, GATORGUARD Jr. or GATORGUARD Table Top Holders. Do not use the sharps container without one of the above items supplied by Tyco Healthcare.

CABINET

To install container in cabinet:

- Unlock and open cabinet door.
- With deposit opening facing you, slide sharps container by handles into cabinet so that sharps container is suspended by both handles resting on ledges near top of cabinet (Figure 1).
- Close and lock cabinet door.

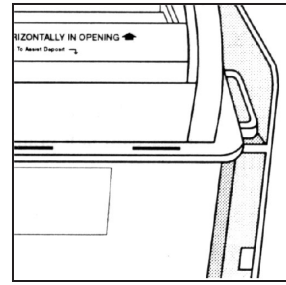
Figure 1

To remove container from cabinet:

Before opening cabinet door, lid must be closed and snapped shut.

TO CLOSE THE LID

- Start with the center lock. Using your thumb, push the lock pad down into the slot. Using your other thumb, push down on the lid above the lock pad until it “clicks” in place. Repeat for the other two locks (Figure 2).
- Unlock and open cabinet door.
- Remove sharps container from cabinet using both handles. Never hold used sharps container below recommended fill line. Keep container in an upright position, then dispose according to facility protocol.
- Slide new assembled sharps container into position.
- Close and lock cabinet door.

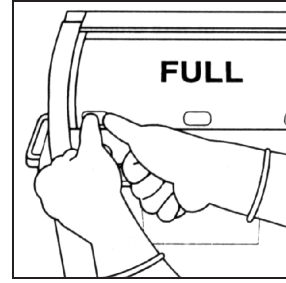
Figure 2

HIDDEN BRACKET

(Used only with 4835 Series Container)

To install container in hidden bracket (Figure 3):

- Insert bracket flanges into slots at back of container top.
- Swing container towards hidden bracket to lock in place.

To remove container from hidden bracket:

Before removing container from hidden bracket, lid must be closed and snapped shut.

- If lid is open, close and snap all three lid locks.
- Insert slotted key into bottom of hidden bracket with slot pointing towards wall (Figure 4).
- Rotate key 90° in either direction.
- Pull key down to release container.

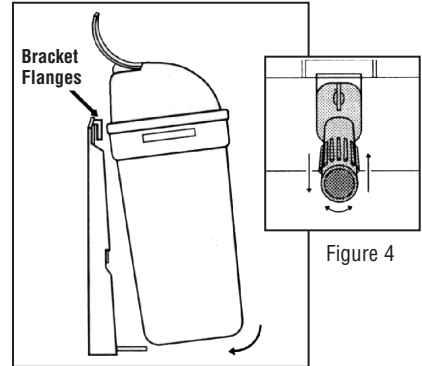
Figure 3

Figure 4

WIRE BRACKETS

To install container into wire brackets (Figure 5):

- Slide sharps container downward into opening of wire bracket until container's top lip catches on bracket.
- For wire bracket with lock: Pull lock bar over top of sharps container and lock.

To remove container from wire brackets:

Before removing container, lid must be closed and snapped shut.

- If lid is open, close lid and snap all three lid locks (Figure 2).
- For wire bracket with lock: Unlock and pull lock bar up over top of sharps container. Remove sharps container from wire brackets using both handles. Never hold used sharps container below recommended fill line. Keep container in an upright position, then dispose according to facility protocol.

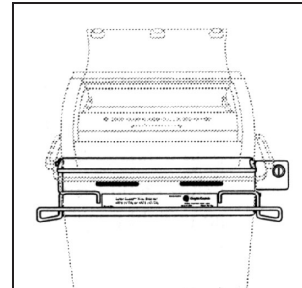
Figure 5

TABLE TOP HOLDER

To install container into table top holder (Figure 6):

- After placing sharps container into table top holder, press down firmly.

To remove container from table top holder:

Before removing container, lid must be closed and snapped shut.

- If lid is open, close lid and snap all three lid locks (Figure 2).
- Remove sharps container from table top holder using both handles. Never hold used sharps container below recommended fill line. Keep container in an upright position, then dispose according to facility protocol.

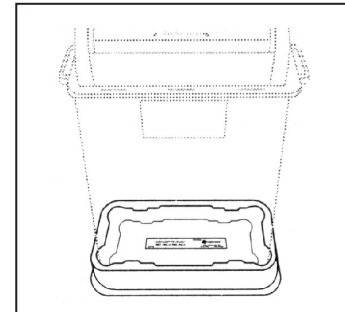
Figure 6

CLEANING INSTRUCTIONS FOR CABINET

Clean cabinet regularly according to the following instruction to maintain appearance and function.

Wipe exterior of cabinet with damp cloth and mild soap or cleaning solution. Never clean with solvents such as acetone or alcohol.

Never attempt to clean sharps container or interior of cabinet when sharps container is installed. Clean interior only when no sharps container is installed.

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